medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium NV
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Canada Inc / médi Canada Inc
 597, Rue Duvernay, Verchères
 QC Canada J0L 2R0
 T: +1 450-583-3317 / +1 800 361 3153
 F: +1 888-583-6827
 service@medicanada.ca
 www.medicana.ca

MAXIS a.s.,
 medi group company
 Slezská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegårdsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi Japan K.K.
 5-1 Nihonbashi Kabutocho Chuoku
 103-0026 Tokyo
 Japan
 T: +81 3 6778 2590
 F: +81 3 5847 7901
 info@medi-japan.jp
 www.medi-japan.co.jp

medi Nederland BV
 Heusing 5
 4817 ZB Breda
 The Netherlands
 T +31-76 57 22 555
 F +31-76 57 22 565
 info@medi.nl
 www.medi.nl

medi Polska Sp. z o.o.
 ul. Łąbiedzka 22
 44-121 Gliwice
 Poland
 T: +48-32 230 60 21
 F: +48-32 202 87 56
 info@medi-polska.pl
 www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
 Rua do Centro Cultural, no. 43
 1700-106 Lisbon
 Portugal
 T +351-21 843 71 60
 F +351-21 847 08 33
 medi.portugal@medibayreuth.pt
 www.medi.pt

medi RUS LLC
 Business Center NEO GEO
 Butlerova Street 17
 117342 Moscow
 Russia
 T +7-495 374 04 56
 F +7-495 374 04 56
 info@medirus.ru
 www.medirus.ru

medi Sweden AB
 Box 6034
 192 06 Sollentuna
 Sweden
 T +46 8 96 97 98
 F +46 8 626 68 70
 info@medi.se
 www.medi.se

medi Trading (Shanghai) Co. Ltd.
 Room 1703, T1, CES WEST BUND CENTER,
 No. 277, Long Lan Road
 Xuhui District
 200232 Shanghai
 People's Republic of China
 T: +86-21 50582319
 F: +86-21 50582319

Medi Türk Ortopedi Medikal İc ve Dis Tic. AS
 Mustafa Kemal Mah. 2156 Sokak No: 16/4-5
 06510 Çankaya Ankara
 Turkey
 T: +90 312 435 20 26
 F: +90 312 434 17 67
 info@medi-turk.com
 www.medi-turk.com

medi UK Ltd.
 Plough Lane
 Hereford HR4 0EL
 Great Britain
 T +44-1432 37 35 00
 F +44-1432 37 35 10
 enquiries@mediuk.co.uk
 www.mediuk.co.uk

medi Ukraine LLC
 Evhena Sverstiuka str, 11.
 Kiev 02002
 Ukraine
 T: +380 44 591 11 63
 F: +380 44 392 73 73
 info@medi.ua
 http://medi.ua

medi USA L.P.
 6481 Franz Warner Parkway
 Whitsett, N.C. 27377-3000
 USA
 T +1-336 4 49 44 40
 F +1-888 5 70 45 54
 info@mediusa.com
 www.mediusa.com

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Important notes

This medical device **MD** is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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Aluminium, Polyester, Elasthan, Elastomer

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Spinomed®

Instructions for the orthopedic technician

Intended purpose

Spinomed is a brace designed to actively relieve load and correct the lumbar spine/thoracic spine in the sagittal plane.

Indications

All indications requiring a relief and/or correction of the lumbar/thoracic spine as well as movement restriction in sagittal and frontal plane:

- Osteoporotic collapse of thoracic and/or lumbar vertebrae
- Juvenile Scheuermann's disease
- Kyphosis with chronic back pain

Contraindications

None known to date

Risks / Side Effects

If the aids are too tight, this can cause local pressure sores or constriction of blood vessels or nerves. Therefore you should consult the doctor treating you before you use it if the following circumstances apply to you:

- Condition or injury of the skin in the area of application, particularly with infectious indications (excessive heat, swelling or reddening)
- Paraesthesia and circulatory disorders (e.g. as seen in diabetes, varicose veins)
- Lymph drainage disorders – equally, swelling of soft tissue adjacent to the area of application which is not related to a condition

When wearing snug-fitting aids, this can result in localised skin inflammation or irritation that is due to mechanical irritation of the skin (particularly in conjunction with sweating) or due to the composition of the material.

Intended patient groups

Healthcare professionals should provide care to the adults and children, applying the available information on the measurements/sizes and necessary functions/indications and in line with the information provided by the manufacturer, and acting under their own responsibility.

Fitting the orthosis (this should be carried out only by an orthopedic technician!)

Wear the orthosis over your clothes.

The back brace does not need to be removed from the pocket during adjustment. Hold the back brace against the patient's back so that the lower edge of the brace is level with the coccyx. The upper edge should reach to approx. 3 – 5 cm below C7. The back brace is formable when cold and can be shaped without a tool. Start by shaping at the lordosis and then work upwards.

Adjusting the straps

Let the patient put the brace on and close the abdominal pad. Then adjust the straps in the following sequence: pelvic strap, shoulder strap, elastic connecting strap. For each strap, undo the Velcro, pull the strap tight, and secure it again. As soon as all the straps have been correctly adjusted, they can be shortened using a pair of scissors.

Pelvic strap

Ensure that the pelvic strap runs below the pelvic bone. This strap should be tight.

Shoulder strap

The shoulder strap should gently hold the patient erect, but should not cut in. Select the tension so that the shoulder strap lies flat against the shoulder.

However, it should still be possible to push the flat hand between the strap and the shoulder. It is important that you stretch the entire belt from top to bottom. Check the fit of the shoulder pad, which should be positioned so that the pad is in contact with the anterior area of the shoulder. Under the arm, the strap should curl up to avoid any chafing.

Every shoulder strap is equipped with a pre-shaped plastic element that makes putting the belt on easier. This can be removed at any time.

Elastic middle strap

Adjust the middle strap to the optimum length so that the reverse buckle is located in the extension of the armpit and the patient's shoulders are adequately held backwards.

Checking the fit of the brace

- Back pad adjusted to fit the patient's spinal curvature
- The pelvic strap runs firmly below the pelvic bone
- The shoulder strap is flat against the body and the flat hand can still be inserted between the strap and the shoulder
- The shoulder pad is in the optimum position
- The elastic connecting strap is adjusted to the correct length

Spinomed®

Dear patient!

Your doctor has recommended that you wear a Spinomed. In order to make application easier, please read the instructions carefully.

Putting on the orthosis

Wear the Spinomed over your clothes. To

make it easier for you to put on the brace, it is helpful to stand in front of a long mirror.

Putting on the Spinomed is like putting on a rucksack

- Undo the large hook and loop fastening at the front (A+B)
- Put on the orthosis as you would normally put on a rucksack. The lighter side of the back brace goes against your back (Fig. 4).
- Pass your right hand through the loop on the right-hand closing side. Hold the left-hand closing side with your other hand and close the tummy sheet. The orthosis is positioned correctly when the closing edge is immediately on top of the batten.

Now look in the mirror to check

- The fit of the back brace: this should nestle closely against your spine. The bottom edge of the brace is level with the buttock cleft (fig. 4).
- The strap system; none of the straps should be twisted.
- The fit of the shoulder pad; the strap must not cut into the underarm area.
- The reverse buckles; these must not press in at the side.

Caution

To avoid the possibility of the back brace slipping, the pelvic strap must be tightened more than the shoulder strap. Slight upward slippage of the brace will not have any effect on the function of the orthosis. For the sake of comfort, however, you should pull the brace down again.

Mode of operation

When you wear the Spinomed, your back muscles will be activated. As a result, your upper body will be gently straightened, countering the curvature of the spine which is typical of

osteoporosis. The Spinomed stimulates the body's own muscle resources. The back splint will have been individually adjusted by your orthopaedic technician. Together with the straps, the brace supports your muscles in holding your shoulders back and straightening your upper body. You will still be able to freely move your shoulders and arms as well as breathe unhindered. If the brace has been optimally adjusted, it can reduce pain and increase day-to-day mobility.

Application

While you are getting used to the brace and to progressively re-establish muscle activity, wearing of the brace should be gradually increased. This means that you should wear the Spinomed a little longer each day, gently training the muscles. Depending on physical condition, an uninterrupted wearing time of 30 minutes will be sufficient at the beginning, which is then gradually increased.

Care instructions

The aluminum splint must be removed before washing. Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Keep the product in a dry place and do not expose to direct sunlight.

Material composition

Aluminum, polyester, elastane, elastomer

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

In the event of any complaints regarding the product such as damage to the fabric or a fault in the fit, please report to your specialist medical retailer directly. Only serious incidents which could lead to a significant deterioration in health or to death are to be reported to the manufacturer or the relevant authorities in the EU member state. The criteria for serious incidents are defined in Section 2, No. 65 of the Regulation (EU) 2017/745 (MDR). The traceability of this product is assured via a UDI code UDI.

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