### ! ATTENTION!

The brace should be used according to the instructions given by the doctor or physiotherapist. When used for the first time, the support should be put on by the experienced health-care personnel. Range of motion should be adjusted according to instructions given by the doctor or physiotherapist. Before the first use, please, read the user's manual carefully. If you feel discomfort while using the brace, in case of swelling or other skin problems, please, consult a doctor or physiotherapist.

### ! ATTENTION!

Not observing the rules of personal hygiene and washing the product with inappropriate detergents may result in skin abrasion or other discomfort related to perspiration and spread of bacteria. Please do not use the product together with warming ointment or gel. After washing, rinse the product carefully - residue detergent left in the product may cause skin vexation and damage of the orthosis fabric.

### ! ATTENTION!

Each and every serious incident related to the product should be reported to the manufacturer and competent authority of the Member State in which the user or patient is domiciled.

PLEASE SEEK MEDICAL ADVICE PRIOR TO USING THIS PRODUCT.

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PL

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EN

SPLINT FOR PALM AND FOREARM WITH THUMB SPICA

CZ

SPLINT FOR PALM AND FOREARM WITH THUMB SPICA

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**Product description:**

A splint for wrist, palm with a thumb hold. The splint assures the functional positions of wrist and hand. The soft hypoallergic pad provides for comfort, fasten with Velcro belts.

**Indications:**

- Wrist, hand and fingers disease which require support and stabilization in specific corrective and/or pain reducing positions. Tendon inflammation, carpal tunnel syndrome, rheumatic joint inflammation, contractures preventive treatment, circumferential nerves injuries, surgery.

**Putting the orthosis on:**

- Please remember to choose the proper size of orthosis to have good treatment.
- Put hand and forearm on the splint and fasten the Velcro belts.

**Maintenance:**

The pad can be washed up by hand in lukewarm water using mild detergents. Do not chlorinate, do not whirlpool. Wring gently before drying. Dry in a distance of artificial sources of heat. Do not iron.

Rozmiar	XS	S	M	L
Width * (cm)	6 - 7	7 - 8	8 - 9	9 - 10

\*hand width at the height of metacarpophalangeal joints.

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