ČEŠTINA

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ENGLISH

CureTape Beauty is a premium-quality elastic kinesiology tape with a hypoallergenic adhesive layer. It is latex-free, water-resistant, contains no medicinal substances, and can be comfortably worn for several days.

INTENDED USE

CureTape Beauty is designed to be applied to the skin. It can stimulate the body's natural healing process. It is also suitable for fixation and targeted support.

INSTRUCTIONS FOR USE:

- Apply to clean, dry skin.
- Gently rub after application for proper adhesion.
- Remove gradually in the direction of hair growth.
- Consider using a tape remover if removal is difficult.

CAUTION:

- Discontinue use if skin irritations or allergic reactions arise.
- Avoid use on damaged skin and on sensitive areas (eyes, mouth).
- Before full application, conduct a sensitivity test with a small piece of tape on the skin to check for adverse reactions.
- Do not store in direct sunlight or near other heat sources.
- Keep out of children's reach.
- For single-use only.

Ecoworks Korea Thysol Hitech Co. Ltd is not liable for any injuries or damages resulting from incorrect use of CureTape Beauty or when used outside its intended purpose. Serious incidents must be reported to the local authorities and the manufacturer.

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