ACTIMO[®] LIBERTY

Gebrauchsanweisung | Instructions for use | Mode d'emploi | Istruzioni per l'uso
| Instrucciones de uso | Gebruiksaanwijzing | Návod k použití

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ACTIMO® LIBERTY HMV 10.50.04.1273

01 WICHTIGE INFORMATIONEN

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INSTRUCTIONS FOR USE ACTIMO® LIBERTY

01 IMPORTANT INFORMATION

General information	12
Product labelling	12
Intended purpose and product description	12
Indication	12
Contraindication	12
Declaration of conformity	12

02 PRODUCT CONSTRUCTION

Scope of delivery	13
Structure of the Actimo® Liberty	13

03 PRODUCT IN OPERATION

Assembly	14
Using the brakes and parking brakes	15
Changing the position of the pole holder clamp	15

04 SAFETY INSTRUCTIONS

Safety instructions	16
---------------------------	----

05 INSTRUCTIONS FOR USE

Cleaning and disinfection	16
Storage	16
Maintenance	16
Re-use	16
Disposal	16

06 MANUFACTURER'S DECLARATIONS

Service life	17
Warranty	17
Notification / Incident	17

07 TECHNICAL DATA

Dimensions and technical information	17
--	----

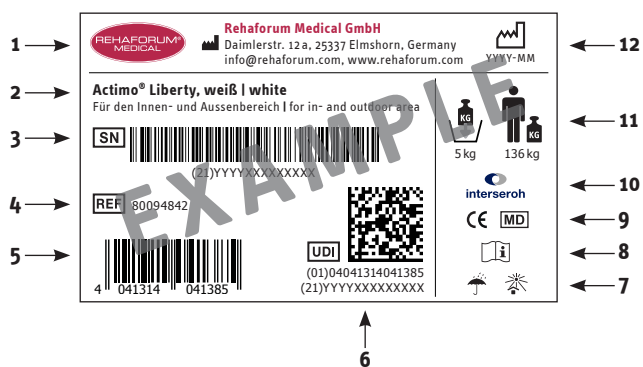
GENERAL INFORMATION

Thank you for choosing a rollator from Rehaforum.

Please read the complete instructions for use carefully before installing, adjusting or using the rollator for the first time. This will provide you with important information that will allow you to fully utilise the technical advantages. In addition, you will receive information that will ensure operational safety and the best possible maintenance of your rollator. If you require further information about the rollator, please contact your specialist dealer.

These instructions for use contain all the necessary information for handling the rollator. Please keep them in a safe place so that they are available for future reference.

PRODUCT LABELLING

1. manufacturer | 2. article description | 3. serial number | 4. article number | 5. EAN code | 6. UDI = Unique Device Identifier, (01) UDI-DI/GTIN, (21) serial number | 7. rain protection symbol / sun protection symbol | 8. observe instructions for use | 9. CE marking / medical device | 10. dual system | 11. max. load | 12. production date

INTENDED PURPOSE AND PRODUCT DESCRIPTION

The rollator is intended to support walking and maintain activity for people with limited mobility. The rollator is suitable for both indoor and outdoor use. The rollator has a seat, steerable front wheels, a stick holder, on all sides reflectors and a carry bag for transporting utensils or shopping. The seat allows the user to take breaks. The rollator can be folded up to save space by pulling up the seat. The height of the handles can be adjusted. To ensure safety in road traffic, the rollator is equipped with a total of nine reflectors. These are located on the seat (two pieces) in parallel alignment with the rear wheel, on the side of the height adjustment frame (two pieces), above the Bad (Reflekto Strip on the Flap and on the front wheel forks (two pieces on each side) to ensure a sufficient reflective effect.

INDICATION

The use of the rollator is suitable for persons with

- slight restriction of movement due to, for example, joint diseases or after operations
- slight loss of strength due to cardiovascular disease, for example
- slight balance problems

CONTRAINDICATION

The use of the rollator is not suitable for persons with

- perceptual disorders
- severe balance disorders
- severe movement restrictions due to, for example, joint damage to the arms or loss of limbs on both arms
- severe visual impairment

DECLARATION OF CONFORMITY

Rehaforum Medical GmbH declares the conformity of this product in accordance with EU Regulation 2017/745.

SCOPE OF DELIVERY

The following components and accessories are included in the scope of delivery

- Actimo® Liberty including back strap
- pole holder
- bag
- Instructions for use

STRUCTURE OF THE ACTIMO® LIBERTY

- | | | |
|---|---|------------------------------------|
| 1. Handle | 8. Reflectors on the seat | 15. Brake cable |
| 2. Brake handle | 9. Seat | 16. Rollator frame |
| 3. Brake adjustment screw | 10. Release strap to fold up the rollator | 17. Tilting aid |
| 4. Back strap | 11. Side reflectors | 18. Rear wheel |
| 5. Fastening of the back strap | 12. Fixation of the folding function | 19. Cane holder |
| 6. Push handle, height adjustable with step indicator | 13. Bag | 20. Wheel fork with reflector |
| 7. Locking mechanism for height adjustment | 14. Reflector strips | 21. Front wheel (swivelling wheel) |

ASSEMBLY

Ask your specialist dealer for assistance with assembly and adjustment. The Actimo® Liberty rollator is delivered assembled and folded in a cardboard box/polybag. Check the packaging and contents to ensure that they are intact and complete.


1. Remove the rollator from the box and place it on the floor. Unfold the rollator by pushing the seat down.

2. Attach the handrail tube and adjust the handle height to the correct position using the handrail adjustment knob. When adjusting, do not pull the entire handle tube out of the frame and make sure that the handles are at the same height!

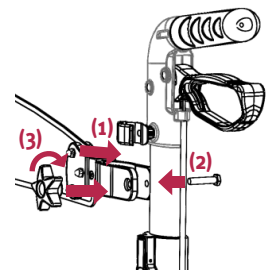3. To attach and adjust the backrest, insert the plastic part of the backrest around the handrail tube (1). Now adjust the backrest to the correct position. To fasten the backrest, insert the hexagonal screw through the plastic part (2) and tighten it with the star knob (3).

4. Attach the bag to the suspension device on the seat using the suspension loops.

5. To fold the rollator, pull the seat upwards by the belt handle.

NOTE The rollator is equipped with a tilting aid on both sides, which enables you to easily overcome an obstacle or an edge, for example. When using the rollator, please ensure that you have a secure footing. Now press lightly on the tilting aid with the tip of one foot so that the front wheels are raised slightly. Now push the rollator onto the obstacle and lower the front wheels forwards again. You can now remove your foot or the tip of your foot from the tilting aid. Please note that using the tilting aid can affect your balance. It is therefore imperative that it is only used if the obstacle is not too high and you have a secure footing.

USING THE BRAKES AND PARKING BRAKES

1. To activate the brake (parking brake), press the brake levers down. To release this again, pull the brake levers back up. **ALWAYS** use both parking brakes at the same time to keep the Actimo® Liberty stationary!

NOTE ON PARKING POSITION

Make sure to park on a flat, non-slip surface and ensure that the parking support is engaged before sitting down. Sit up straight and do not lean back too much.

2. To brake briefly or to slow down, pull the brake levers up at the same time so that the Actimo® Liberty does not turn sideways. Depending on the speed and condition of the ground, the intensity of the braking force can be adjusted.

CHANGING THE POSITION OF THE POLE HOLDER CLAMP

1. Take the supplied Allen key, position it on the screw and unscrew it.
2. Also loosen the screws on the other side.
3. Insert the screw into the pole holder clamp as shown.
4. Fit the pole holder clamp and screw the other side tight as well.

EN

SAFETY INSTRUCTIONS

- Only use the rollator if it is in perfect condition and ensure that the rollator is completely unfolded.
- Do not use a defective rollator!
- The ends of the handles must be parallel to the floor. Check that the wing screws are tight and that the height is set correctly before using the rollator!
- Check the brakes after each height adjustment and make sure that the brake cables are not pinched.
- Test the brakes before each use and always activate the right and left brakes at the same time to brake!
- Pay attention to the wear of the tires! Heavily worn tires can impair the rollator's braking effect. Have damaged parts replaced by your specialist dealer.
- A rollator is not a wheelchair, but only a walking aid. Do not allow yourself to be pushed when sitting on the padded seat!
- Always sit on the seat with the brakes on!
- Always sit slowly on the seat and do not throw yourself against the backrest! Make sure the backrest is fastened before each use!
- Do not sit on the seat if the back belt is not attached or the rollator is at an angle.
- The rollator must not be used as a seat in vehicles such as trains or buses!
- Only use the rollator on level surfaces – under no circumstances on stairs or on high edges! To ensure stability and avoid accidents, all four wheels must always be in contact with the ground during use.
- Always hold on to both handles to maintain balance.
- The rollator must not be carried on removable parts or brake cables.
- Be careful when unfolding and folding the rollator. There is a risk of pinching!
- Do not hang bags on the handles. There is a risk of tipping over!
- Do not overload the rollator (max. luggage load 5 kg).
- Make sure that the loaded bag has an influence on the maneuverability of the rollator and do not walk backwards when using the rollator!
- Maximum user weight is 136 kg.
- Do not make any design changes to the rollator that you have not previously discussed with your specialist dealer or Rehaforum Medical GmbH!

CLEANING AND DISINFECTION

Clean and disinfect the rollator at regular intervals, e.g. daily. Before passing the rollator on to another person, it must be thoroughly cleaned and disinfected. Use a mild soapy water solution without abrasive additives and a soft cloth. Disinfection can be carried out using commercially available agents in accordance with the manufacturer's instructions. Do not use abrasive or corrosive cleaning agents or high-pressure or steam cleaners! Make sure that no dirt is deposited on the tyres or brakes. This can be removed with a soft brush and warm water. Also make sure that no cleaning fluid runs into the frame tubes to avoid corrosion!

STORAGE

Protect the rollator from dust and moisture and store it in a safe place to protect the frame and wheels from damage! Protect the rollator from strong heat sources.

MAINTENANCE

The rollator should be serviced annually by your specialist dealer. The stability of the frame, the strength of the mountable parts and the tightness of all screws must be checked and repaired if necessary. All screws must be tightened. Worn, damaged or broken parts must be replaced immediately.

 **Functional faults** If you notice a reduction in braking force, sluggishness of the wheels, uneven running of the wheels or other faults such as rust on the linkage, contact your specialist dealer. Do not use a defective or damaged rollator!

RE-USE

The rollator is suitable for re-use. Before passing the rollator on to another user, it must be checked, cleaned, disinfected and serviced by a specialist dealer. If parts of the rollator are damaged or destroyed, they must be replaced with new ones. It is recommended that the wheels are also replaced each time the rollator is used. All loosened screws and nuts must be tightened. The rollator must be in perfect condition before it is used again.

 **It is forbidden to re-use the rollator if irreparable damage to the rollator, such as a bent frame, is detected! The instructions for use must be handed over with every change of user.**

DISPOSAL

Do not dispose of the rollator with normal household waste. Please contact your specialist dealer or take the rollator to your local waste disposal centre.

SERVICE LIFE

If the rollator is used for its intended purpose and is cared for, checked and maintained in accordance with the instructions, a service life of five years can be assumed. However, the service life can vary considerably depending on the handling and care of the rollator. For example, particularly careful handling of the rollator can extend the service life, while extreme and improper handling of the rollator can shorten the service life. The determination of the service life by Rehaforum Medical GmbH does not constitute an additional guarantee.

WARRANTY

The warranty period is 24 months and applies to all product defects that are demonstrably attributable to manufacturing or material defects. Excluded from the warranty are Damage caused by natural wear and tear or intent. Furthermore, the warranty does not cover damage caused by incorrect cleaning and care products or damage caused by negligent and improper operation or use.

NOTIFICATION / INCIDENT

Any serious incident that has occurred in relation to the rollator must be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

DIMENSIONS AND TECHNICAL INFORMATION

Item no. 80094842 | **Colour** white
Item no. 80094843 | **Colour** alpine green
HMV 10.50.04.1273
Weight incl. bag and backrest 7.3 kg
Weight without bag and backrest 6.3 kg
Width 66.5 cm
Height 75.5 - 92.4 cm
Depth 69.6 cm
Seat width 44 cm
Seat height 55 cm
Seat depth 20 cm
Height of the backrest 63 - 81 cm
Width of the handles 3.3 cm
Width between the handles (center line) 52.5 cm
Folded size (width x height x depth) 69 x 78 x 25 cm
Recommended height 160 - 190 cm
Wheel thickness 3.6 cm
Wheel diameter 20 cm
Max. turning diameter 90 cm
Max. load 136 kg
Max. load bag 5 kg
Material frame anodized aluminum (not susceptible to corrosion)
Material wheels Polyurethane (PU)
Material handles thermoplastic elastomers (TPE)
Material seat / bag / backrest 100% Oxford cloth

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