2

3

6

7

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INSTRUCTIONS FOR USE EN

Dear patient,

We ask you to carefully follow the enclosed instructions for use. If you have any questions, please contact the doctor looking after you, your nearest specialist store or us directly.

INTENDED USE

NEURODYN® COMFORT is a foot lifter brace with semi-elastic 8-way restraint for dynamic compensation of incomplete and spastic paralyses.

INDICATIONS

• Flaccid and spastic foot lifter paresis (e.g. after stroke, multiple sclerosis, peroneal paresis, other neurological/muscular diseases)

CONTRA-INDICATIONS

- Allergic, inflammatory or injury-related skin changes (e.g. swelling, redness) of the body areas to be supplied.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenic disorders of the sensory system and skin trophism in the body area to be treated (sensory disorders with or without skin damage).

SIDE EFFECTS

If used appropriately and correctly, there are no general side effects, intolerances with long-term effects or allergic reactions known to date.

INTENDED PATIENT TARGET GROUP

The target group is all patients, taking into account indications and contraindications. Instruction and allocation of the correct size is carried out by medical professionals.

PUTTING IT ON AND USE

- We recommend wearing a stocking under the brace.
- Place the brace in your shoe, let the lifting reins look out to the right and left **1**.
- Position the flat ankle brace centrally over the Achilles tendon, with the Y-shaped rein pointing outwards **2**.
- The trick with the click: Gently squeeze the ankle brace until it snaps shut **3**.
Important: Only compress the ankle brace inwards. If it is bent outwards, the sewn-in metal band can break. This must be avoided at all costs.
- Close the ankle brace with the inner Velcro strap **4**.
- The ankle brace can be readjusted via the outer, Y-shaped rein with the deflection eyelet **5**.
- Put on your shoe, place your foot on the ground and bend your knee forward **6**.
- Fasten the lifting straps crosswise and under slight tension to the ankle brace. Test the forefoot lift during a few steps. Adjust the reins if necessary **7**.

NOTES FOR THE ORTHOPAEDIC TECHNICIAN

• The sole of the NEURODYN® COMFORT can be individually adapted to the respective shoe.

⚠ IMPORTANT INFORMATION

- Essentially, the indication for and duration of wearing of any orthopaedic device in general, as well as the presence of any of the conditions below in particular, should be discussed with the doctor who is in charge of your treatment.
- Check the functioning of the product with your doctor or specialist retailer.
- The product is designed to treat one patient.
- Do not wear the product directly against open wounds.
- In order for the product to have a long service life and function, it may not be worn in conjunction with oily/greasy or acidic products, ointments or lotions.
- The product can come into contact with splash water.
- Please always close any Velcro fasteners present properly. Damage caused by improper closing of the Velcro fasteners does not constitute a reason for complaint.
- In the event of serious incidents when using this aid, immediately contact your doctor, your health care supply retailer or contact SPORLASTIC directly at info@sporlastic.de to report the incident.
- Any improper modifications to the product and/or improper use of the above-mentioned product exclude any product liability on the part of the manufacturer.
- Possible health-related mutual risks or other disadvantages with certain treatments that may arise in association with the use of the product must be discussed with the doctor in charge of treatment.

☞ Please return the packaging and the product to the local recycling collection system. Please comply with local regulations.

☞ SPORLASTIC GmbH is affiliated with a returns system for packaging and therefore fully complies with the packaging legislation in the Federal Republic of Germany.

CARE

We recommend washing NEURODYN® COMFORT ankle braces and sling bars gently in lukewarm water (30°C) with liquid detergent and air drying. Do not dry the product in the tumble dryer, on the heater or in the microwave. The insert can be cleaned with a damp cloth.

☞ TRANSPORT AND STORAGE CONDITIONS

Please ensure that the product is stored in a dry place and is protected against moisture and sunlight. Store the product at a normal temperature and humidity.

QUALITY MANAGEMENT SYSTEM

All products from SPORLASTIC GmbH are subject to product inspection as part of our quality management system. If you still have any complaints about our product, we kindly ask that you contact your specialist retailer.

📄 An online version of the instructions for use can be found on our website at www.sporlastic.de.

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